



Easy to use



Lockable with a coin

KOALA

POSITIONING BELT

- **EASY TO USE**
- **QUICK TO INSTALL**
- **LOCKABLE**

The new Koala belt system has an ingenious structure that makes the belt superior in supporting a mobility restricted person in a vehicle and wheelchair.

The five-point Pilot lock makes it possible to lock each latch plate at different times. The shoulder harnesses can be freely adjusted for persons of different sizes. The lock can also be locked using a coin.

The belt system is used with the vehicle's own seatbelts.
Patent pending no. 20125231

Manufacturer:



Autovi Systems Oy
Rovaniemi • Finland
www.autovi.com

Sales:



Quick to install



INSTRUCTION MANUAL

REMEMBER! Always use the vehicle's own seatbelt with the Koala belt system.

Affix the included seatbelt cutter to the dashboard in case of emergency.



1.

Open the package. Remove the belt and open the folds.



2.

Open the top buckle.



3.

Feed the lower strap behind the lower part of the backrest.



4.

Bring the upper strap behind the headrest and attach the lock behind the backrest.



5.

Adjust the belt from behind to achieve the correct restraint.



6.

Open the buckles from the front.



7.

Remove the installation card.



8.

Adjust the belts to the correct lengths. The belt is ready to use.



Autovi Systems Oy
Rovaniemi • Finland
www.autovi.com